

CardioPAL SAVI

FINGER ELECTRODE ACCESSORY

PREPARATION

Insert the finger-electrode accessory into the top of the monitor,

so that the embossed

heart is

upright;



you'll hear a click when it's firmly attached. Keep the monitor in a location that is easy to access.

HOW TO RECORD YOUR ECG

1. When you feel a symptom, press the **Symptom** button on the front of the monitor. Listen for the beep.

2. Moisten your fingertips, and place them on the electrodes (contacts) shown below.



Press down firmly with each finger.

3. Limit activity; watch the time countdown on the monitor display. When you see, "Recording Complete," remove your fingers. You'll hear 2 beeps, and see the Transmit message on the display.

*If you pressed **Symptom** by mistake, just press the **NO** button to stop the recording.*

