

# Your At-Home Heart Monitoring Procedure

Here's what to expect...



For Self-Hookup Event and MCT Procedures  
Using *TelePatch*™ or *Duet*™ Monitoring Systems



**MEDICOMP**

*Advancing diagnostic cardiology since 1981*

## What is this test?

---

Your doctor has ordered a heart monitor from Medicomp to be shipped to your home. It should arrive in the next three to five business days. It is important to hook up to the monitor and wear it as soon as it arrives for the number of days your doctor prescribed. This monitor can help your doctor discover any heart irregularities, called arrhythmias.



## How does it work?

---

The monitor will record your heart's electrical activity. These ECG recordings are wirelessly sent to a Medicomp cardiac monitoring lab via the cellular phone network. Highly trained, certified cardiac technicians analyze the data, and then create reports for your doctor to review and interpret.

### For more information

Visit our online Patient Center at  
[medicompinc.com/patient-center](http://medicompinc.com/patient-center)

# What Happens Next – It's Easy



**1**

In a few days, your Medicomp monitoring kit will arrive.

**2**

You will receive a kit that looks like one of these.



Follow the instructions included in the kit and watch the video at [medicompinc.com/patient-center](http://medicompinc.com/patient-center)



Medicomp team members are available around the clock if you need help. See the back cover.

**3**

You will wear your monitor 24 hours a day up to 30 days, as prescribed by your doctor. Go about your normal daily activities.



**4**

Promptly return the equipment to Medicomp.



A prepaid return envelope is included in the kit.

**5**

Follow up with your doctor.



Your physician can share the test results with you.

## Don't Delay in Starting

---




***IMPORTANT: Hook up to the monitor and start monitoring your heart as soon as your kit arrives.***

The sooner you use the monitor, the sooner your doctor will begin to receive information to review and interpret.

It's easy to start! Just follow the instructions in the Quick Start Guide included in the monitor kit and the online video. If you need help, we'd be happy to assist you over the phone.

## Don't Delay When Finished

---

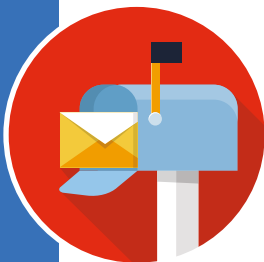

It is imperative to return the monitor to Medicomp using the prepaid return envelope included in the kit as soon as your procedure ends. Non-returned devices could delay data delivery to your doctor and result in a bill to you for the missing equipment.

# About Medicomp

---



We have been advancing diagnostic cardiology since 1981. Medicomp develops, manufactures, and provides service with the most reliable and sophisticated ambulatory heart-monitoring systems in the world. Since its inception, Medicomp has been at the forefront of the cardiac telemedicine industry.

## How to Get Help

---



[medicompinc.com/patient-center](https://medicompinc.com/patient-center)



[patientsupport@medicompinc.com](mailto:patientsupport@medicompinc.com)



877-996-5553